

# EAR VAC EAR WAX CLEANER

---

## **CAUTION:**

- **CHECK WITH YOUR PEDIATRICIAN BEFORE USING ON SMALL CHILDREN.**
- **NEVER USE WITHOUT THE SILICONE ATTACHMENTS.**
- **WASH THOROUGHLY BEFORE AND AFTER EACH USE.**
- **DO NOT MIX OLD AND NEW BATTERIES.**
- **DO NOT MIX ALKALINE, STANDARD, OR RECHARGEABLE BATTERIES.**
- **IF BATTERY CONTENTS ARE LEAKING, DISCONTINUE USE AND DISCARD.**

## **BATTERY INSTALLATION:**

1. Remove the battery cover (located on the side of the item) by sliding it downward.
2. Insert 2 AA batteries matching polarities inside the battery compartment and replace the battery cover.

## **USE INSTRUCTIONS:**

1. Wash your hands thoroughly.
2. Slide a silicone attachment (large diameter in first) onto the small tube located in the front. Four different color silicone attachments are provided for multiple users.
3. Slide the ON/OFF switch to the ON position.
4. Insert the tube with the silicone attachment into your ear, slightly turn the item to help draw ear wax out.  
**DO NOT INSERT THE SMALL TUBE COMPLETELY IN YOUR EAR.**

## **CLEANING INSTRUCTIONS:**

1. Place silicone attachments in warm soapy water.
2. Unsnap the front piece with the small tube from the main unit and place in warm soapy water.
3. Remove the screen (by pulling away from the main unit) and place in warm soapy water.
4. Scrub each piece using the small brush provided to remove any debris.
5. Dry each piece thoroughly using a dry cloth before replacing.