



Pill Organizer Instructions



Pill Organizing

1. Remove pill cutter (C) from tray. Place pill box (B) into the pill cutter tray. Pour a supply of pills onto the left side, then slide pills over the top of the organizer (A) into the pill box (B).

Pill Cutting

2. Place a pill on the pill cutter cavity (D) and close the cutter to split the pill. Note: for best results, align the pill's half mark with the pill cutter's blade.

Stacking

Slide the bottom rails of the top pill organizer (E) through the small hooks on the top of the bottom organizer (A).

Cleaning

Occasionally clean with a damp cloth. Note: never use solvents or abrasive cleaning agents.

